ROBOTIP™ G2

Surgilum, LLC Model TPSG2 (RoboTip G2) for exclusive use with Surgilum, LLC Model RMSG2 (RoboMarker G2) – sold separately

Instructions for Use

Congratulations! Thank you for purchasing your Box of RoboTip G2. Please read these instructions carefully to achieve the best possible results.

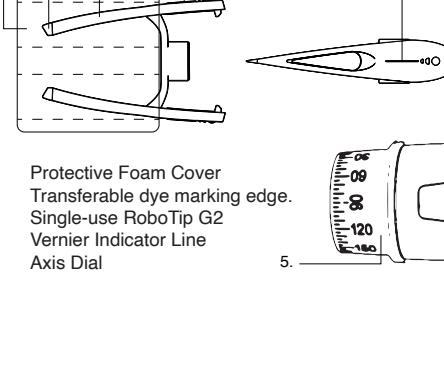
Description: The single use TPSG2 RoboTip is used with the reusable RMSG2 RoboMarker Handle to apply reference marks at a prescribed axis angle to the surface of the cornea or adjacent corneal/scleral junction. The handle maintains marking tip angular position using an enclosed pendulum system and includes a fixation light, projected laser target alignment lines, and an axis dial brake to aid in corneal axis mark placement.

Indications for Use: The RoboMarker G2 (RMSG2) and the RoboTip G2 (TPSG2) are used in combination as a corneal marking system. The system places highly accurate marks on the cornea as well as the peripheral corneal/scleral junction for Toric Intracocular Lens alignment, Femto-Laser Cataract Surgery, Limbal Relaxing Incisions, and other refractive ocular procedures.

Specifications:

- Physical: 25mm X 39mm X 8mm, 1g
- Materials: Proprietary dry ink formula of Gentian Violet (Crystal Violet) and Carbon on a Polycarbonate Carter Tip and a XLPE protective cap.
- Packaging: Poly/Paper sealed pouch with tear notches.
- Mark Geometry: Dots – 1.2mm lines spaced by 7.5mm internal optical zone
- Sclera – 1.2mm lines spaced by 12mm internal optical zone
- Sterility: Radiation, Gamma, ISO 11137 VDmax25

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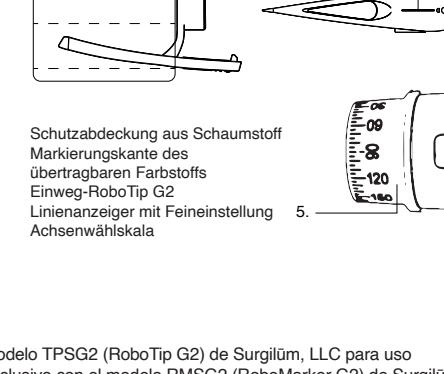
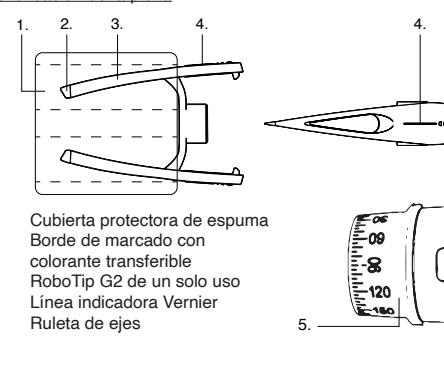
Contraindicaciones: DO NOT USE ROBOMARKER G2, ROBOTIPS G2, WITH ANY PATIENT THAT HAS SENSITIVITY OR ALLERGY TO GENTIAN VIOLET (CRYSTAL VIOLET) DYE.

ONLY USE THE G2 ROBOMARKER SYSTEM ON HEALTHY CORNEAS. DO NOT USE ON INFANTS.

CAUTION:

- The RoboMarker G2 should only be used by healthcare professionals.
- The RoboMarker G2 and RoboTip G2 are precision instruments and must be handled with care to prevent damage that will adversely affect marking accuracy.
- Only use RoboMarker G2 and RoboTip G2 for its intended use and as directed within these instructions.
- The Single-use RoboTip G2 is sterile via Gamma Radiation and intended for single use ONLY. Do NOT attempt to re-sterilize or re-use on more than one patient.
- Do NOT use RoboTip G2 if the sterile pouch is damaged.

Compatibility: The RoboTip G2 is only compatible with the RoboMarker G2 handle (produced and sold separately by Surgilum, LLC). Do not attempt to use the RoboMarker G2 or RoboTip G2 with other marking systems.

CAUTION: RoboTip G2 is a light-sensitive product. Protect from light exposure until the day of use. The RoboTip G2 is a STERILE component that should never be re-used.

Marking Posture: The RoboMarker G2 handle is a pendulum/gravity-based device that must be held HORIZONTALLY for accurate marking. The RoboTip G2 must be inserted on the RoboMarker G2 handle in accordance with the product instructions. Once the RoboTip G2 is properly inserted on the RoboMarker G2 handle, the single-use RoboTip G2 is directed towards the patient who should be in an upright position with their head similar to an upright tilt lamp position (or positioned like it was when the patient's K Readings were obtained).

CAUTION: RoboMarker cannot be oriented vertically such as with a patient in a supine position. Any attempt to do so will result in inaccurate mark placement due to ocular cyclotorsion and loss of pendular gravitational effect.

Inserting a single-use RoboTip G2 into the RoboMarker G2:

NOTE: The RoboTip G2 is a 2-position marking tip. The shifted position is the peripheral corneal/scleral marking position. To achieve an inner corneal marking position, press down between the marking edges after the tip cylinder has been seated into the G2 axial dial receptacle.

1. Open the pouch and remove the single-use RoboTip

2. G2 by the Protective Foam Cover (1). Do not remove the Protective Foam Cover at this time as it may contaminate the dye marking edge of the G2 tip.

3. Insert the stem of the RoboTip G2 into the receptacle in the center of the Axis Dial (5).

4. Remove the Protective Foam Cover (1)

5. Initial placement of the RoboTip G2 onto the G2 achieves the peripheral corneal/scleral marking position.

To achieve an inner corneal marking position, press down between the marking edges; AVOID TOUCHING THE MARKING EDGES (2).

6. Place the RoboMarker G2 handle with inserted G2 tip in your dominant hand leaving your index finger free.

7. Using your index finger, press and hold the Mode Button on the RoboMarker G2 for 3 seconds in order to illuminate the Axis Dial. The device is in "Axis Dial Mode".

8. Use your thumb and forefinger to stabilize the axis dial to rotate the RoboTip G2 until the Vernier line indicator on the RoboTip G2 is aligned with the desired axis degree on the now illuminated Axis Dial.

DO NOT USE THE AXIS BRAKE BUTTON THAT IS NEAR THE AXIS DIAL TO STABILIZE THE DIAL DURING ROTATION OF THE ROBOTIP. DOING SO WILL DAMAGE THE BRAKE FUNCTION OF THE AXIS DIAL BRAKE.

Corneal Marking Procedure: ENSURE YOUR PATIENT'S EYE IS NUMBER ONE! Even if the patient recently received numbing drops, additional numbing drops are highly recommended just before corneal marking with the RoboMarker G2 marking system.

With the RoboMarker G2 Handle on, use your index finger briefly press the Mode Button to place the unit into "Laser Target Mode". This will project the laser target and the Mode and Brake buttons will flash in 1 second intervals.

NOTE: The RoboMarker G2 has an accelerometer

9. Place the RoboMarker G2 handle with inserted G2 tip in your dominant hand leaving your index finger free.

10. Inform the patient that you will be using a blinking laser fixation target along with other instruments that may be used in their surgery.

11. You may want to inform your patient they might feel momentary mild pressure but that they MUST continue to look at the fixation light.

12. Instruct the patient to blink the eye once to moisten the cornea. Then use your other hand to gently hold open the patient's eyelid.

13. Position the RoboMarker G2 at cheek level below the patient's eye.

14. While lifting the G2 from below the eye to be marked up to a position approximately 2 inches (5cm) from the center of the cornea, instruct the patient to look at the 10-linking centric dot of the laser target. The projection of the laser target on the eye will allow you to fine-tune the centration of the RoboMarker G2 tips over the cornea as the marking tips approach the cornea.

CAUTION: If the marking tips do not land simultaneously, a rotation or a "pirouetting" of the marking tips may occur on touchdown. To prevent this, the RoboMarker G2 includes a forward Axis Dial Brake button.

15. After marking the cornea, keep the eyes open for 10 seconds to allow the dye to set. Then instruct the patient to keep both eyes CLOSED until surgery to avoid blinding off the marking dye.

(Placing a light towel over both eyes of the patient will discourage blinking and help the marks to remain defined.)

16. Squeeze front of tip together to remove single-use RoboTip G2. Turn off the RoboMarker G2 Handle by pressing and holding the Mode Button for 3 seconds.

17. Safely dispose of used tips in accordance with your waste procedures.

Note: RoboTip G2 tips are recyclable.

Storage and Cleaning: Unused RoboTip G2s must remain sealed in their original sterilized pouch and stored in a dry area in the original open pack box as the box label includes important tip batch number and the expiration date information.

Environmental Limitations

Use +10C -35C (50F - 95F), <90% RH

Storage & Transportation -10C -35C (14F - 130F), <95% RH

Maintenance & Service: The RoboTip G2 is a single use device and should not be re-used. There is no maintenance/service required for the RoboTip G2.

CAUTION: Single-use RoboTips G2 should not be used after the expiration date printed on the over pack box.

CAUTION: Single-use RoboTips G2 CAN NOT be reused. Any attempt to do so may cause harm or injury.

Recommendation: Practice using the RoboMarker G2 before surgery on a conventional object of similar size to

detect whether the G2 handle is level. When the projected target is "steady" (not flashing) the G2 handle is slightly out of linear alignment however, reasonably good marks may be obtained but note that these marks may have a slight "A" or "V" configuration.

If the projected laser target is blinking, the device is slightly out of linear alignment however, reasonably good marks may be obtained but note that these marks may have a slight "A" or "V" configuration.

NOTE: The projected laser target will turn off if the G2 handle is not level enough to obtain reasonably accurate corneal marking.

9. Hover your index finger above the brake button.

10. Inform the patient that you will be using a blinking laser fixation target along with other instruments that may be used in their surgery.

11. You may want to inform your patient they might feel momentary mild pressure but that they MUST continue to look at the fixation light.

12. Instruct the patient to blink the eye once to moisten the cornea. Then use your other hand to gently hold open the patient's eyelid.

13. Position the RoboMarker G2 at cheek level below the patient's eye.

14. While lifting the G2 from below the eye to be marked up to a position approximately 2 inches (5cm) from the center of the cornea, instruct the patient to look at the 10-linking centric dot of the laser target. The projection of the laser target on the eye will allow you to fine-tune the centration of the RoboMarker G2 tips over the cornea as the marking tips approach the cornea.

CAUTION: If the marking tips do not land simultaneously, a rotation or a "pirouetting" of the marking tips may occur on touchdown. To prevent this, the RoboMarker G2 includes a forward Axis Dial Brake button.

15. After marking the cornea, keep the eyes open for 10 seconds to allow the dye to set. Then instruct the patient to keep both eyes CLOSED until surgery to avoid blinding off the marking dye.

(Placing a light towel over both eyes of the patient will discourage blinking and help the marks to remain defined.)

16. Squeeze front of tip together to remove single-use RoboTip G2. Turn off the RoboMarker G2 Handle by pressing and holding the Mode Button for 3 seconds.

17. Safely dispose of used tips in accordance with your waste procedures.

Note: RoboTip G2 tips are recyclable.

Storage and Cleaning: Unused RoboTip G2s must remain sealed in their original sterilized pouch and stored in a dry area in the original open pack box as the box label includes important tip batch number and the expiration date information.

Environmental Limitations

Use +10C -35C (50F - 95F), <90% RH

Storage & Transportation -10C -35C (14F - 130F), <95% RH

Maintenance & Service: The RoboTip G2 is a single use device and should not be re-used. There is no maintenance/service required for the RoboTip G2.

CAUTION: Single-use RoboTips G2 should not be used after the expiration date printed on the over pack box.

CAUTION: Single-use RoboTips G2 CAN NOT be reused. Any attempt to do so may cause harm or injury.

Recommendation: Practice using the RoboMarker G2 before surgery on a conventional object of similar size to

the cornea. Many surgeons will use a grape. Limited Warranty: RoboMarker G2 and single-use RoboMarker G2 Tips are warranted to be free of defects in materials and workmanship. If any defects are discovered please notify Surgilum, LLC.

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