



PIL-MA LOADING SHEET

677P/677PY/677PMY

Medjet PIL-MA injection system

3! PIL-MA
FLEX | HYDROPHILIC PRELOADED


Single use, sterile, disposable device (injector) for the implantation of a foldable intraocular lens into the eye. The device consists of two parts: the assembled injector and the cartridge.

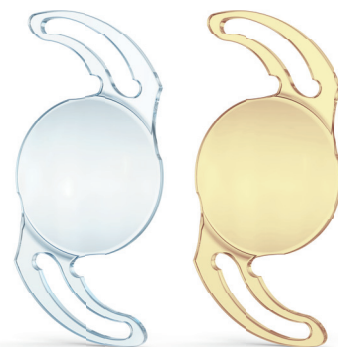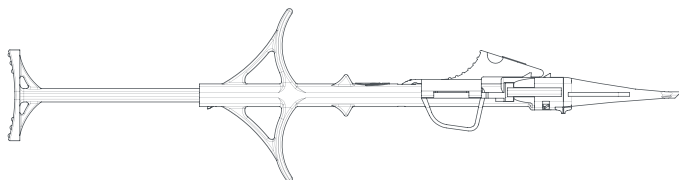

1.

1st CLICK LOADING

1. Insert a drop of viscoelastic material into the lens holder.

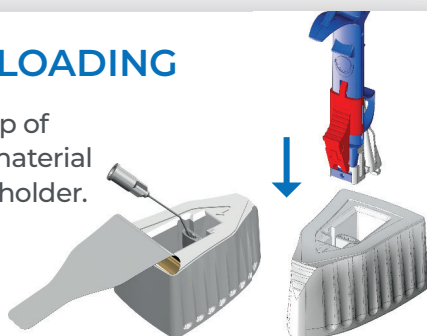


2. Insert the injector into the wet container, until you hear a 'click'.

2.

2nd CLICK CLOSING


3. Inject viscoelastic material into the cartridge nozzle

4. Fold up the cartridge nozzle

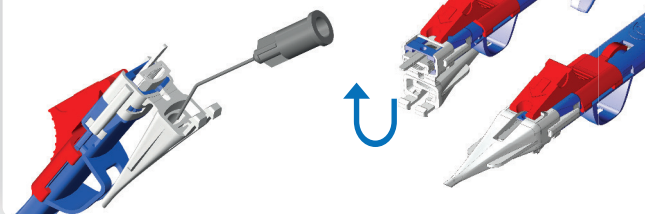

3.

3rd CLICK READY

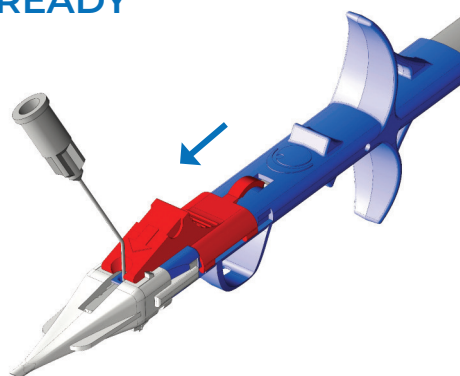


5. Push the red stopper forward until it clicks into place.

6. Fill viscoelastic material into the small opening on the top of the cartridge nozzle.

4.

INJECTION

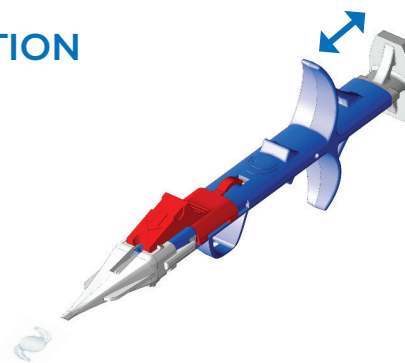

7. Advance the plunger until midway, until the lens enters the nozzle.

Pull the plunger back to the end, to separate the IOL from the silicone tip. You are ready for injection. Push the plunger forward again, until the IOL leaves the nozzle.

For detailed information please see Instructions for Use (IFU) of the product: medicontur.com/eifus

MEDICONTUR

Medicontur
Medical Engineering Ltd
export@medicontur.com
www.medicontur.com

Head Office
Herceghalmi út 1.
2072 Zsámbék
Hungary

International Office
Chemin du Champ-des-Filles 36
1228 Plan-les-Ouates
Switzerland / Geneva

MA-IOL-PILMA-MD057-INF-EN